

Packing Sand Rider in the box

Supplies needed: (available at U-Haul, box stores and office supply stores)

1/8" foam: 2' wide x 25', shrink wrap, 2" packing tape

To insure returns without damage, **Sand Rider should be arranged for boxing and then shrink wrapped BEFORE placing in the box for shipping.** Placing the Sand Rider in the box is a two person lift.

1. Pad all edges of the Sand Rider, especially axle tubes and front end receiver tubes with foam. Lay down 2 strips of shrink wrap about 8 feet long, side by side. (you will lift and stretch these to shrink wrap the bottom when finished) Center a strip of foam 2' by 3' on top of the shrink wrap. Lay one of the large Sand Rider wheels at one end of that foam.

2. Position the seat base with the axle tube to the right as shown. Place prongs toward the edge of the foam (and eventually the corner of the box) Place foam collar on brake knob (if equipped.) Pad in between with a layer of foam.

3. Lay the backrest on the seat base with the logo down. Pad in between with a layer of foam.

4. Turn the front section upside down and place it on top of the seat base. Wheel should nestle in the handle. Pad in between with a layer of foam.

5. Place remaining tire on top toward center, as flat as possible.

6. Pull up the shrink wrap strips from step 1 and stretch over the tire that is on top. Wrap shrink wrap several times around the assembled Sand Rider and over the top.

7. Place the shrink wrapped Sand Rider in box (two person lift).

8. Pad armrests and axles and place them in the box. Put all knobs and pins in the zip lock bag and place in box. Tape securely.

