

Seatbelt Installation as an Add-On

Tools required:

You will need two 3/8" wrenches

1. Remove the drop nose pins from the rear axles, slide out the rear wheels and set them aside. Lift out the armrests and set them aside. If possible, place the rear axle on a box for easier access.
2. Using the 3/8" wrenches, remove the bolts that held up the armrests.
3. Arrange the seatbelt with the quick release cam lock positioned for attachment on the user's left hand side (for right hand release). Verify that the top of the seatbelt will face upwards.
4. Add a nylon washer to the 3 1/2" stainless bolts. Slide the bolt through the mounting hole of the seatbelt bracket and add another washer to the bolt.
5. Push the bolt through the top hole of arm rest mounting tube from the outside inward. Add another washer and the lock nut to the bolt where it exits the tube. Tighten snugly with the wrenches.
5. Repeat 4-5 on the other side of the seat base.
6. Verify proper positioning and function of the seatbelt by tugging on it while closed and then releasing the cam lock.



7. Reassemble the Sand Rider.